

Dos & Don'ts for wearing P2/N95 Masks

What they are and when to use them

P2 or N95 MASK: A mask which provides a tight facial seal with a face-seal leakage of < 10 % and ability to filter at least 95% of particles 1 micron in size.

TYPES OF P2/N95 MASK







WHEN TO USE:

- · Whenever Airborne Precautions are indicated e.g. TB, measles or chickenpox.
- · When in the operating theatre when laser or diathermy is used during a procedure.
- When in the same room as an induced sputum is being undertaken.

DO

DON'T

DON'T give this type of mask to a patient unless medically directed e.g. chemo.

DO clean your hands before putting on and after DON'T wear if wet of soiled. removing mask. DON'T reuse a mask. Discard it DO follow manufacturer's instructions for donning and doffing the mask. immediately after use. DO check that there no defects such as holes or torn DON'T wear a mask that you haven't been trained to use. Ask for instruction. DON'T use a mask if air leaks around the DO make sure the mask is completely covering your nose and mouth edges. DO mould the mask over the bridge of your nose and DON'T use this mask if you have a beard press around the face seal of the mask. 2mm or more in length, it will not seal. DO ensure mask has a complete seal all round and DON'T touch the front of your mask after perform a fit check. it has been correctly fitted. DON'T snap the straps of the mask as DO tilt your head forward and remove the mask by bringing the bottom strap to the top strap, controlling this may cause contamination. the elastic, lift straps over the your head and without DON'T leave a mask hanging around touching the front of the mask remove the mask out, down and away. your neck as this can lead to cross infection.

Developed by ISLHD IMACS April 2018

Based on APIC Do's & Don'ts For wearing N95 respirators in non-surgical healthcare settings 2017

ISLHD DOC 231 Trim: DT18/59988 REVISION 0 April 2018 Page 1 of 1